- For post-curing of the parts, please select the parameter settings for your process. You can find more qualified printers on our website www.dreve.dentamid.de.

- Post curing:

PCU LED N₂
Rapid Shape D, DII / Asiga MAX UV 8 minutes, 80 % light power (vacuum)

FotoDent® flash
Rapid Shape D, DII / Asiga MAX UV 8 minutes + 1 minute gas pre-flow

RS Cure
Rapid Shape D, DII / Asiga MAX UV 8 minutes, 75 % light power (vacuum)

- Impurity due to the construction or a break of the material and impurity due to operation mistakes cannot be excluded. Thanks to the low viscosity however it is possible to filtrate FotoDent® setup. It is recommended to regularly take out the container of the production unit, to homogenize and filtrate the content. Upcoming entrained bubbles get eliminated through a holding time of approx. 1–2 hour with FotoDent® setup.

To avoid damage of material quality do not expose the liquid material to light under any circumstances. Variations from the described manufacturing process may lead to modified mechanical characteristics and/or colour variations of the FotoDent® setup material.

Possible dangers

Please note the safety instructions for processing FotoDent® setup.


Hazard statements: **H318** Causes serious eye damage. **H317** May cause an allergic skin reaction. **H412** Harmful to aquatic life with long lasting effects. **H360D** May damage the unborn child.

Precautionary statements: **P261** Avoid breathing dust/fume/gas/mist/vapours/spray. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P310** Immediately call a POISON CENTER/doctor. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

Contains: 2-Propenoic acid, reaction product with Pentaerythrite, 1,4 Butanediol dimethacrylate, 2-benzyl-2-dimethylamino-4-morpholinobutyrophenone, phenyl bis(2,4,6-trimethylbenzoyl)-phosphineoxide, Pigments

Lot number / Expiry date

The lot number and the expiry date are indicated on each FotoDent® setup packaging. In case of claims please always indicate the lot number of the product. Do not use the product after the expiry date.

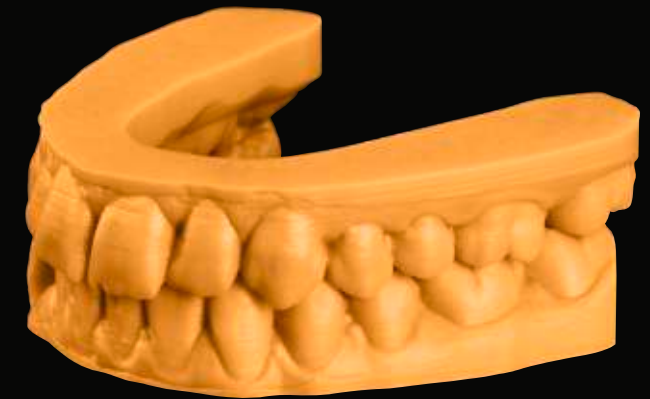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Intended use

FotoDent® setup is a material based on (meth-)acrylate resin for DLP systems with 385 nm / 405 nm LED for the production of dental models. For the post curing of FotoDent® setup built parts, the light curing units PCU LED N₂, FotoDent® flash and RS Cure are recommended, which are optimal balanced to the production process.

Processing

- During the application of FotoDent® setup we recommend wearing personal protective equipment e. g. suitable gloves, safety goggles etc.
- FotoDent® setup should be well shaken approx. 1 hour before use.
- Please see further steps within printer manual.
- Carefully fill pour FotoDent® setup into the pre-determined container of the production unit.
- Remove all bubbles with a cleaned object.
- Processing temperature:
Rapid Shape D and DII printer 23 ± 3 °C
Asiga MAX UV 35 ± 3 °C
- Select the product specific parameter settings for FotoDent® setup for your process.
- When the building process is finished a direct post treatment is recommended.
- After the platform is taken up a dripping off time of approx. 10 minutes is recommended.
- Resin coated parts get clean with isopropanol (97 %) within a time of about 2 x 6 minutes. For a particular effective cleaning with isopropanol the use of explosion-protected ultrasonic units is recommended. Alternatively, cleaning with isopropanol can be carried out in a separate beaker under pouring conditions for approx. 2 x 6 minutes. Recommendation: Beaker and magnetic stirrer, laboratory shaker or ultrasonic cleaner. In case of insufficient cleaning (such as shiny areas on the object after drying) repeat cleaning process.

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